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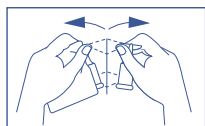
CONFEZIONE

10 contenitori monodose richiudibili da 0,5 ml

Acuviss eye drops is a **hydrating, protective and humectant** sterile solution for those who suffer from dry eyes (poor lacrimation), and is able to relieve eye fatigue and redness caused by various factors including:

- exposure to **environmental conditions** (wind, intense and prolonged direct or indirect solar radiation, pollen, etc.)
- exposure to **aggressive agents** (pollution, smog, dust, salt or chlorinated water, etc.)
- **intense visual concentration** for study, work, prolonged computer use, driving for long periods
- **use of contact lenses**

INSTRUCTIONS FOR USE

Open the protective pouch and remove a single-dose bottle from the strip. Open the single-dose bottle by turning and then lifting the cap until it is removed. Refit the cap with the closing pin facing upwards. Raise the head and, keeping the lower eyelid gently lowered with a finger, apply 2-3 drops of solution in the conjunctival sac by lightly pressing the vial, every 4/5 hours. If necessary, repeat the operation in the other eye. Close the vial immediately after use: the contents can be reused within the following 12 hours. Acuviss eye drops can also be used by wearers of contact lenses. Acuviss eye drops are intended for frequent use multiple times per day. In compliance with the dosage indicated above. The single-use product is for single-patient use.

DOSAGE

Apply 2-3 drops per eye every 4/5 hours. We recommend alternating treatment cycles with suspensions of 7 days. Each cycle should not exceed 30 days of treatment.

COMPOSITION

Purified water, Hydroxypropylmethylcellulose (0,2%), Euphrasia officinalis flower/leaf/stem water (10%), Calendula officinalis flower water (10%), Vaccinium myrtillus leaf extract (10%); Sodium Chloride (0,77%); Sodium phosphate monobasic monohydrate; Disodium phosphate.

CONTRAINDICATIONS

The medical device is not suited for subjects with known hypersensitivity to the components in the formula.

WARNINGS

Before use, carefully read the information provided by the manufacturer. Do not exceed the recommended dose. Discontinue treatment in the event of any undesirable effects.

Do not use in conjunction with other topical treatments. Do not swallow. The product is intended for ophthalmic use only. The product is single-use and single-patient; any residual solution must neither be reused nor shared. Store in a dry place, away from light and heat, at a temperature between 5° and 25° C. Do not use the product if the packaging is open or damaged.

Do not use the product after the expiry date. The expiry date refers to the unopened and properly stored product. The product is intended for adult patients. Keep out of sight and reach of children. Hands should be thoroughly washed before applying the solution. When administering the solution, be careful not to touch the eye and conjunctiva membrane with the tip of the vial. Do not dispose of the container in the environment after use.

Any serious accident resulting from the use of the medical device must be promptly reported to the manufacturer and the Competent Authority of the member State in which the event occurred.

PACKAGING

10 x 0.5 ml single-dose resealable containers

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